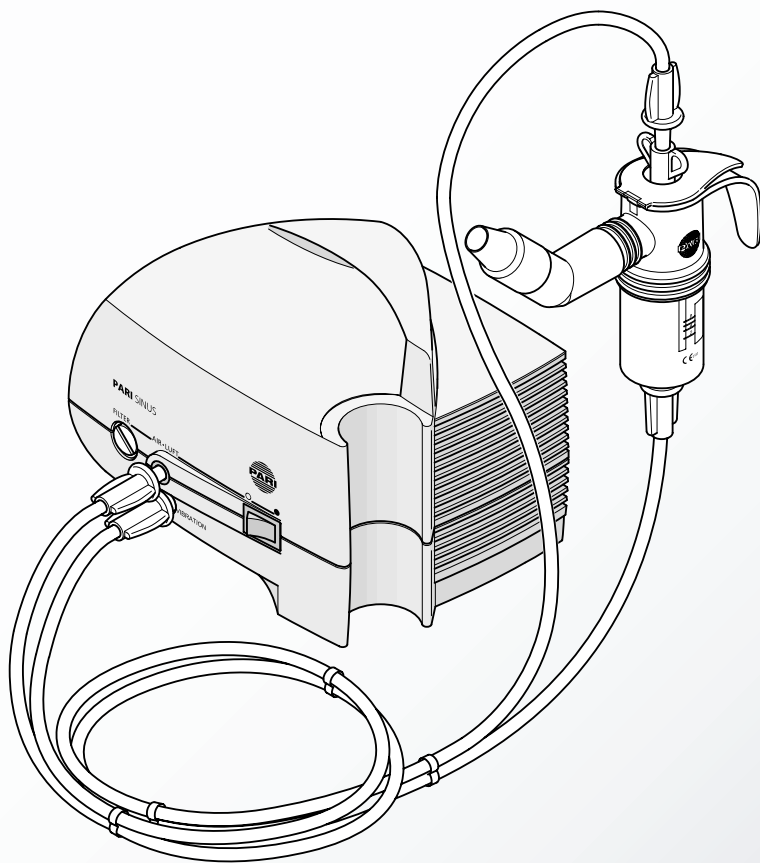
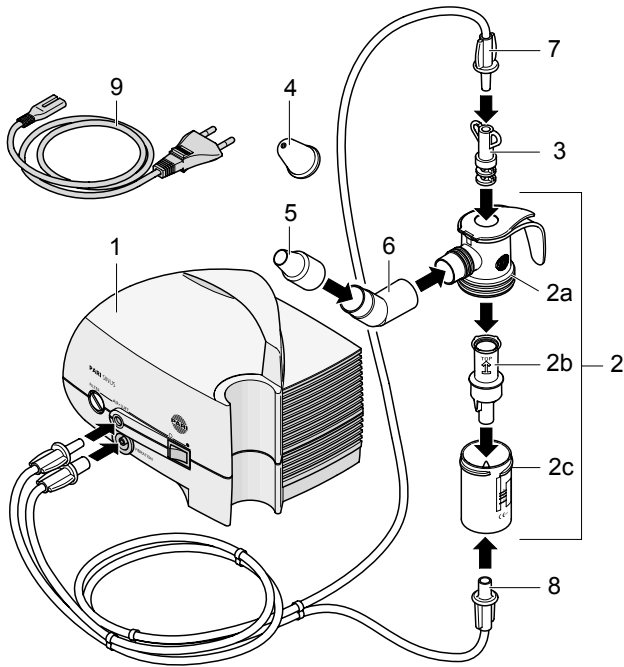
SINUS

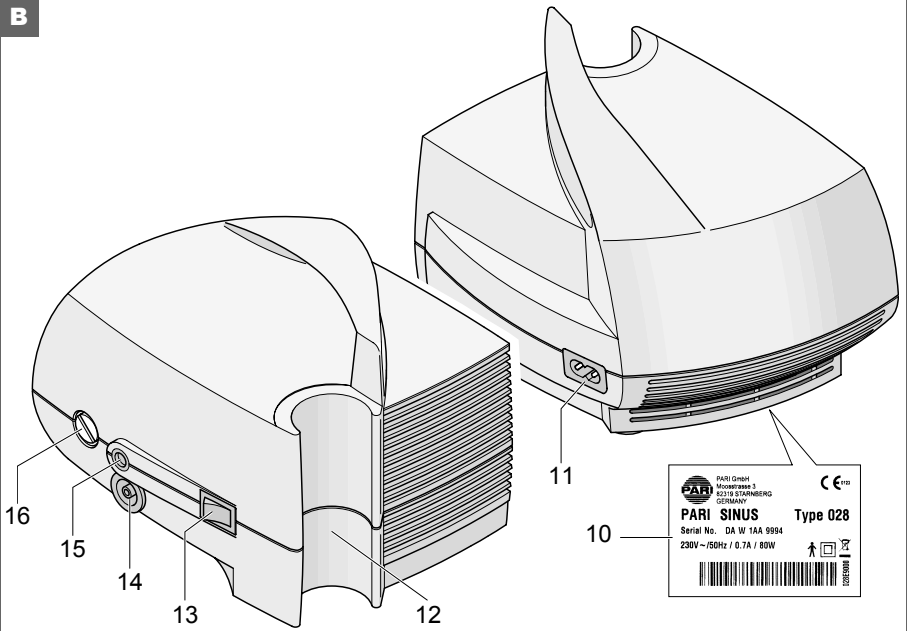
CE 0123

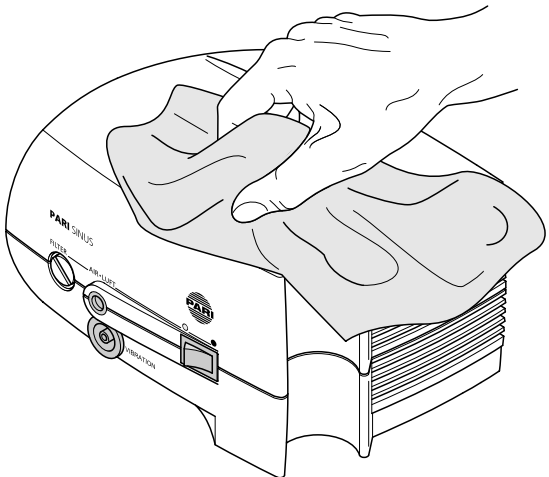
A

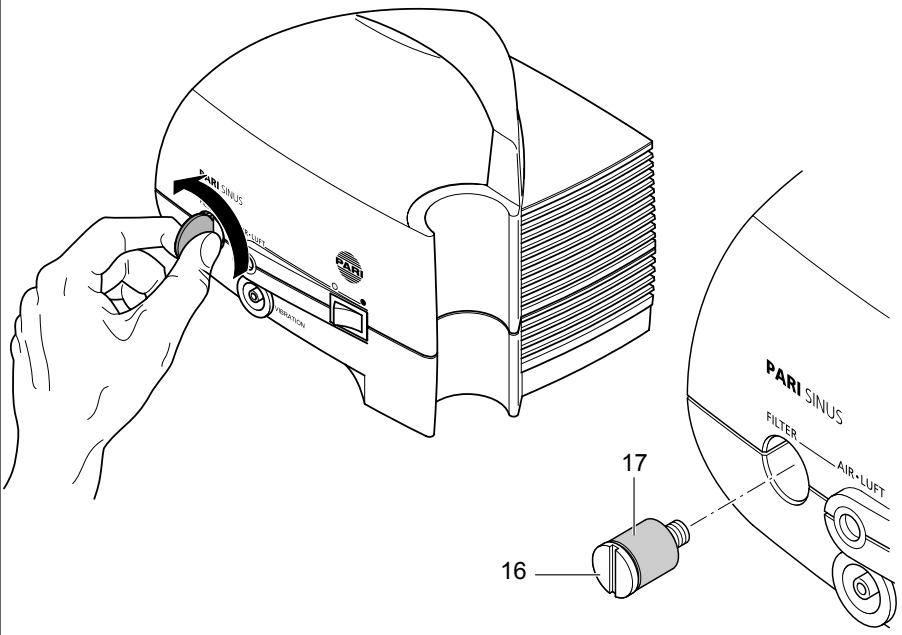
B

C

D

DE	Gebrauchsanweisung	1
EN	Instructions for use	13
DA	Brugsanvisning.....	25
ES	Instrucciones de uso	37
FR	Notice d'utilisation.....	49
IT	Istruzioni per l'uso	61
NL	Gebruiksaanwijzing	73
NO	Bruksanvisning	85
PT	Manual de Instruções	97
SV	Bruksanvisning	109

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Read these instructions carefully and completely before using for the first time. Keep these instructions in a safe place.

Risks caused by failure to comply with the instructions:

Failure to comply with the instructions may cause personal injury and/or damage the product. Specifically, failure to follow the instructions may have the following dangerous consequences:

- Failure of important functions of the product – resulting in inefficient treatment
- Danger of personal injury from heat, electrical, mechanical or other effects – in all variations and with all associated consequences

Table of contents

1	Components	14
2	Working parts	14
3	Important notes	15
4	Intended use	16
5	Information	17
6	Preparation for use	18
6.1	Connecting the PARI LC SPRINT SINUS nebuliser	18
6.2	Connecting a PARI LC PLUS or LC SPRINT nebuliser	19
7	Cleaning	20
7.1	Compressor	20
7.2	Care of tubes	20
8	Filter change	21
9	Disposal	21
10	Technical data	22
11	Shipping and storage	23
12	Terms and conditions of warranty	23
13	Explanation of symbols	24

1 Components

Please check that your PARI SINUS compressor (028G1000) contains all the following components (see Figure A):

(1) PARI SINUS compressor	(4) Nose plug
(2) PARI LC SPRINT SINUS nebuliser	(5) Nasal joining piece
2a Nebuliser upper section	(6) 45-degree knee
2b Nozzle attachment (orange)	(7) Bolus tube for vibration
2c Nebuliser lower section	(8) Pressure hose
(3) Vibration connection	(9) Power cord

2 Working parts

The PARI SINUS compressor includes the following working parts (see Figure B):

(10) Type identification plate (bottom of device)	(13) On/off switch
	(14) Vibration connection for bolus tube
(11) Power cord socket (rear of device)	(15) Air connection for pressure hose
	(16) Filter holder
(12) Holder for nebuliser	

3 Important notes

The PARI SINUS compressor is an electrically operated appliance. It must not be left unattended while it is in operation, or used in damp areas or where there is a risk of explosion.

Whenever the appliance is used, a thorough knowledge of the instructions for use and compliance with the same is assumed. In all cases, the operator is responsible for the safe operation of the appliance if it is not used appropriately for its intended purpose.

The appliance may only be serviced by PARI GmbH or a service center expressly authorised by PARI GmbH. If repairs are carried out by an authorised PARI GmbH service center, you must ask for a certificate giving the date, type and scope of the repair, and the company name and signature. Only original PARI accessories may be used to operate the appliance.

Please remove the power plug from the socket to ensure that the power is completely cut off.

Safety instructions:

- If the device or the power cord is damaged, or if a fault is suspected after the device has been dropped or suffered a similar shock, the system must not be operated and/or the power plug must be disconnected immediately.
- The power plug must be disconnected
 - if a malfunction occurs during operation,
 - before cleaning and maintaining the appliance,
 - after each use.
- Do not pull the power plug out of the socket with wet hands; never pull on the power cord to disconnect the plug.
- Do not run the power cord over sharp edges or allow it to be pinched; do not let it hang unsupported and do not expose it to excessive heat or oil.
- Babies, infants, children, and anyone who requires assistance must be supervised constantly by an adult during inhalation therapy. Otherwise, safe and effective treatment cannot be guaranteed.
- Anyone standing below the compressor may be injured if it falls.

If PARI inhalation devices are rented or re-used, please follow the guidelines for preparing PARI inhalation devices for re-use when changing patients issued by the manufacturer. These guidelines are available upon request from PARI GmbH, or on the Internet at www.pari.de.

PARI GmbH is not responsible for damage or malfunctions caused by the operator using the machine improperly or contrary to the instructions.

4 Intended use

When used in conjunction with a PARI LC SPRINT SINUS nebuliser, the PARI SINUS compressor forms a highly effective inhalation device for treatment of the upper respiratory tract with nebulised medications. It is suitable for temporary¹⁾ nasal inhalation of liquid medication prescribed or recommended by a doctor for inhalation treatment of the upper respiratory tract.

The PARI SINUS compressor can also be used with any other PARI LC PLUS or LC SPRINT nebuliser.

In this combination, the PARI SINUS compressor forms a highly effective inhalation device for treatment of the lower respiratory tract with nebulised medications. It is suitable for temporary²⁾ oral or nasal inhalation of liquid medication prescribed or recommended by a doctor for inhalation treatment of the lower respiratory tract.

Using nebulisers and accessories made by other manufacturers can cause the inhalation device to be used improperly, which will impair the efficiency of the treatment because the PARI compressor and the PARI nebuliser are designed to complement each other.

- You must always have a medical examination before treating illnesses.
- You must only use medication that has been prescribed or recommended by your doctor.
- Patients who respond to the treatment with adverse reactions should stop the therapy.
- Following surgery or where inflammation of the middle ear is present, a risk/benefit assessment for vibrating aerosols should be carried out by the doctor before the start of treatment.

1) Each application lasts less than 1 hour, in this case max. 5-8 minutes.

2) Each application lasts less than 1 hour, in this case max. 20 minutes.

5 Information

When treating acute and chronic inflammation of the nasal sinuses (sinusitis) with medication in aerosol form, special requirements are necessary for generation and application of the aerosol. The aim is to apply the medication through the ostia (connections between the main nasal cavity and nasal sinuses) directly to its site of action, the nasal sinuses. To achieve this aim, the appliance generates a "vibrating aerosol" that is conveyed more easily through the ostia to the nasal sinuses. The advantage of such topical therapy over oral, systemic application (e.g., in tablet form) is that less medication is required. Thus it can be expected that there is a reduction in the occurrence of side effects. This also moistens the mucous membranes of the nose.

The treatment must not be performed when the nose is blocked. Before starting the treatment, make sure that the nose is decongested e.g., by flushing with the PARI MONTESOL nasal douche and the PARI MONTESOL nasal solution. If necessary, use products designed to reduce swelling prescribed or recommended by a doctor.

6 Preparation for use

- The device should be at room temperature when it is used.
- Place the PARI SINUS compressor on a flat, clean and dust-free surface.
- Do not run the compressor on carpets or in very dusty environments, e.g., under the bed, in workshops etc.
- Ensure that the side ventilation slits are not blocked.
- The compressor must not be covered under any circumstances during operation.
- Check that the supply voltage complies with the data marked on the appliance.

6.1 Connecting the PARI LC SPRINT SINUS nebuliser

Treatment of the upper respiratory tract only with PARI SINUS 028G1000

Figures A and B:

- Insert the adapter of the pressure hose (8) into the "Air" connection (15) of the compressor, twisting slightly.
Attach the other end to the PARI nebuliser (2).
- Insert the adapter of the bolus tube (7) into the "Vibration" connection (14) of the compressor, twisting slightly.
- Insert the other end of the bolus tube into the "Vibration" connection (3) of the nebuliser.
- Connect the nasal joining piece (5) to the knee (6) and attach the other end to the PARI LC SPRINT SINUS nebuliser upper section (2a).
- Connect the compressor (1) to the mains power supply:
 - Plug the power cord (9) into the socket (11) on the compressor.
 - Insert the mains plug of the power cord into a suitable socket.

To disconnect the PARI compressor from the mains after the treatment session, follow the above steps in reverse order, pulling the plug out of the socket **first**.

- Before treatment, pull the nebuliser upwards out of the nebuliser holder (12).
- Turn the compressor on at the On/Off switch (13).
- After treatment, put the nebuliser back in its holder.

For connection, use of the PARI LC SPRINT SINUS nebuliser for the first time and the procedure for therapy, read the instructions for use supplied with the nebuliser.

6.2 Connecting a PARI LC PLUS or LC SPRINT nebuliser

Treatment of the lower respiratory tract with PARI SINUS compressor 028G1010

Figures A and B:

- Insert the adapter of the pressure hose (8) into the "Air" connection (15) of the compressor, twisting slightly.
Attach the other end to the PARI nebuliser (2).
- Connect the compressor (1) to the mains power supply:
 - Plug the power cord (9) into the socket (11) on the compressor.
 - Insert the mains plug of the power cord into a suitable socket.

To disconnect the PARI compressor from the mains after the treatment session, follow the above steps in reverse order, pulling the plug out of the socket first.

- Before treatment, pull the nebuliser upwards out of the nebuliser holder (12).
- Turn the compressor on at the On/Off switch (13).
- After treatment, put the nebuliser back in its holder.

In this inhalation method, the tube for vibration (7) is not needed and the vibration connection (14) for the bolus tube is not occupied.

For connection, use of the PARI nebuliser for the first time and the procedure for therapy, read the instructions for use supplied with the nebuliser.

In order to avoid any risk to health (e.g., an infection), please also read the Chapter "Hygienic re-use" in the instructions for use of the PARI LC SPRINT SINUS or LC SPRINT nebuliser. It is also important that you observe the subsequent instructions on "Cleaning" and "Filter change" in these instructions.

7 Cleaning

7.1 Compressor

Before cleaning, turn the compressor off and remove the power cord from the socket.

Do not pull the cable to remove the plug from the socket.

Figure C:

- Wipe the outer surface of your PARI SINUS compressor housing with a damp cloth.

Do not spray liquids into the ventilation slits. Such liquid may damage the electrical equipment and other compressor parts and cause a malfunction.

If liquid gets into the compressor, please contact an authorised dealer or the PARI Service Center (Tel.:+49-(0)8151-279 220 or info@pari.de). Do not use your PARI SINUS compressor except on their instructions or after a check and any necessary repairs have been performed by your authorised dealer or the technical Service Dept. of PARI GmbH.

7.2 Care of tubes

Depending on the environmental conditions, condensation may occur in the pressure hose and the bolus tube during usage. To ensure that treatment is carried out properly condensation must be removed after use.

To do this proceed as follows:

- Detach the pressure hose (8) and the bolus tube (7) from the nebuliser.
- Leave the pressure hose inserted in the compressor (compressed air connection (15)).
- Leave the compressor running until any condensation in the tubing has been removed by the air-flow.
- Detach the pressure hose from the compressor.
- Now connect the bolus tube with the right end to the compressed air connection of the compressor and proceed as described above until any condensation has also been removed here.

Replace the PARI SINUS tube system (part. no. 041B4570) with every change of patient, or if the tube looks dirty.

8 Filter change

The filter on the front plate of your PARI SINUS compressor must be replaced every 200 operating hours in normal operating conditions, or at least once a year.

Also please check the filter at regular intervals (after every 10 to 12 inhalations). If it is soiled (grey or brown colour) or blocked, replace it. The filter must also be replaced with a new filter¹⁾ if it has become damp.

Do not try to clean the filter and use it again.

Only use original PARI filters; otherwise your PARI compressor may be damaged or treatment may be inadequate.

To change the filter, proceed as follows (see Figure D):

- Unscrew the filter holder (16) from your PARI SINUS compressor counter-clockwise using a suitable coin.
- Pull the old filter (17) out of the filter holder and fit the new filter in its place.
- Insert the filter holder with the new filter in your PARI SINUS and screw it in firmly, turning clockwise with a coin.

9 Disposal

This product falls within the scope of the European Council Directive on Waste Electrical and Electronic Equipment (WEEE²⁾) and is included in Product Category 8: Medical Devices for the purposes thereof. Accordingly, this product must not be disposed of with domestic waste. The disposal regulations prevailing in the respective member countries must be observed (e.g., disposal by local authorities or dealers).

Materials recycling helps to reduce the consumption of raw materials and protect the environment.

1) Part. no. 041B4852, available from your dealer or PARI GmbH.

2) Directive 2002/96/EC of the EUROPEAN PARLIAMENT AND THE EUROPEAN COUNCIL of January 27, 2003 on waste electrical and electronic equipment.

10 Technical data

General

Compressor type:	Double compressor: Piston for generation of compressed air Diaphragm for generation of bolus
Electrical connection:	See type identification plate on the bottom of the compressor
Maximum A-evaluated sound pressure level of the compressor:	approx. 59dBA (acc. to: EN 13544-1:2004-09 section 26)
Operating pressure with LC SPRINT SINUS nebuliser:	150kPa (1.5bar)
Bolus pressure amplitude, absolute:	20kPa (0.2bar)
Frequency of bolus pressure amplitude:	44Hz
Housing dimensions:	(W x H x D) 19.2cm x 14.5cm x 15.0cm
Weight:	1.8kg
Flow:	4.5l/min

Classification according to EN 60601-1:1996 for PARI SINUS

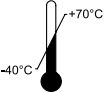
Type of electric shock protection:	Protection class II
Degree of protection from electric shock of the applied part:	Type B
Degree of protection against water penetration in accordance with IEC 529 (IP rating):	No protection
Degree of protection when used in the presence of flammable mixtures of anaesthetics with air or with oxygen or nitrous oxide:	No protection
Operating mode:	Continuous operation

Information concerning electromagnetic compatibility

- Portable and mobile high-frequency communication devices such as mobile phones can disrupt electrical medical equipment.
- The electrical field generated when the compressor is running can interfere with screens and monitors (flickering). If you notice any signs of interference (such as flickering) while the device is in use, move it farther away from other electrical equipment.
- Technical electromagnetic compatibility data is available in table form upon request from PARI GmbH, or on the Internet at www.pari.de, on the "Products/ Inhalation devices/Technical data" tab.

11 Shipping and storage

	Ambient temperature -40°C to $+70^{\circ}\text{C}$
RH 10% - 95%	Relative humidity 10% to 95%
P 500hPa-1060hPa	Atmospheric pressure 500hPa to 1060hPa

12 Terms and conditions of warranty

During the warranty period, we or our customer service will repair any defects in the compressor resulting from faults in material or workmanship free of charge. The owner shall not be entitled to cancel the sale or demand a partial or full refund of the purchase price. This warranty does not cover damage caused by improper use of the appliance. The warranty shall become void if repairs are carried out on the appliance by unauthorised persons. No compensation will be paid for consequential or direct damage.

In the event of a claim under this warranty, please bring the complete appliance to your dealer or send it to us, packed in the original carton and postage paid, together with this warranty certificate.

Wearing parts are excluded from the warranty.

The warranty period shall commence on the date of purchase.

13 Explanation of symbols

	- Caution, observe the instructions for use - Warning
	On/Off
	Alternating current
	Protection class of the part used: Type B
	Protection class II appliance
	The medical device was distributed commercially after August 13, 2005. This product must not be disposed of with normal domestic waste. The symbol of the refuse bin with a cross through it indicates that it must be collected separately.

We reserve the right to make technical changes.

For all product information please contact our Service Center:
Tel.: +49-(0)8151-279 220

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Certificate of Guarantee for PARI SINUS

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We grant a 3 year guarantee on this appliance, commencing on the date of purchase.

In the event of a claim under this guarantee, please bring the complete appliance to your dealer or send it to us, well packed and postage paid, together with this certificate of guarantee.

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Confirmation of purchase: The appliance with the stated appliance number was sold in its original packaging.

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